


TD-5053


AJ Safety Lancet

FEATURES


- 5 depth settings
- Single-use only
- Lancet automatic retraction

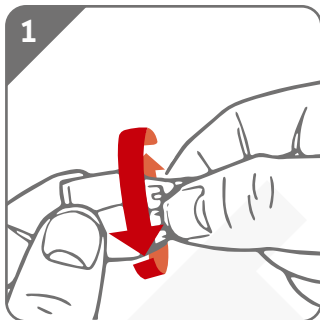


1
Rotate the regulating wheel to choose a proper puncturing depth from 1 to 5.

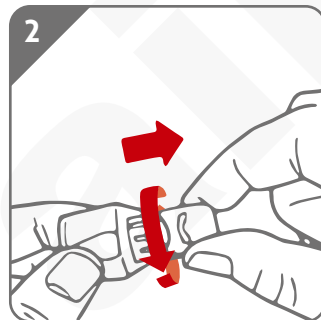

2
Twist off the protective cap.

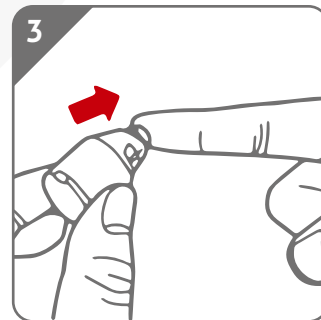

3
Press firmly on the fingertip to get a blood sample.

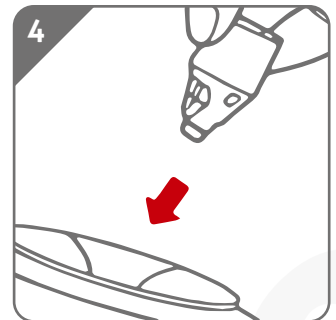

4
Dispose of the used lancet in a sharp container.

Gauge	26G				
Penetration Depth	1.2 mm	1.5 mm	1.8 mm	2.1 mm	2.4 mm
Blood Volume	1-10 μ L	5-20 μ L	10-30 μ L	15-40 μ L	20-50 μ L
Packaging	50 pcs/box				



www.taidoc.com

TaiDoc Technology Corp.

B1-7F., No. 127, Wugong 2nd Road, Wugu Dist.,
New Taipei City 24888, Taiwan
Tel : +886-2-6625-8188 Fax : +886-2-6625-0288